

NON WOVEN ADHESIVE DRESSING ROLL

(Permeable non woven synthetic adhesive tape B.P)



PRODUCT DESCRIPTION :

Permeable Non woven Synthetic Adhesive Tape B.P. is made of a spun laced non-woven textile material spread evenly with a polymeric adhesive mass which does not offset when the tape is unrolled and which does not separate from the backing under normal conditions of use. The tape is permeable to air and water vapour. The adhesive surface is protected with silicone paper.

APPLICATION :

- Retention of post-operative wound dressings.
- To secure catheters and drainage tube.
- Securing dressings and ostomy pouches to fragile skin.
- Retaining gauze and absorbent pads.
- On areas where swelling or movement is expected.
- Wound dressing specially in difficult curved body areas or fragile skin.

FEATURES

Allows joint to be dressed without constraining movement.

Provides reliable fixation for catheters drainage tubes.

Coated with hypoallergenic adhesive.

Excellent cross and diagonal stretch for mild to moderate compression.

Combines security with comfort.

Highly Conformable.



Available Sizes

Code No.	Size cm x mtr	Dressing per Pack
PD 1060	5 x 2.5	1 Roll
PD 1061	7.5 x 2.5	1 Roll
PD 1062	10 x 2.5	1 Roll
PD 1063	5 x 10	1 Roll
PD 1064	7.5 x 10	1 Roll
PD 1065	10 x 10	1 Roll

PINAPORE[®]-ID

Non Woven Island Wound Dressing with Pad



Product Description

Non-woven Island Adhesive Dressing with Pad is a sterile, hypoallergenic, post operative, conformable adhesive dressing consisting of a breathable soft cloth backing with a central non-adherent absorbent pad. This is an all-in-one, sterile, cost effective dressing for wounds. This conforms well to body contours.

Application

This is used to provide protection for closed, clean surgical wounds, incisions, excisions, donor sites, etc which have low to moderate exudates. It can also be used as a secondary dressing over hydrogels, alginates, gauze or other primary dressings or as a tracheostomy tube dressing, catheter/cannula site cover.

Features

- ✓ **Conformable:** Soft and conformable dressing ensures patient comfort. Flexible non-woven layer does not restrict bodily movements when in place
- ✓ **Highly absorbent pad:** Dressing consists of a highly absorbent pad that maximizes passage of blood and exudate into the dressing, minimizes the risk of fluid strikethrough
Low adherent wound contact layer: The pad has a low adherent wound contact layer which minimises disruption or trauma to the underlying tissue on removal of the dressing, reducing the risk of pain on removal
- ✓ **Rounded corners:** Rounded corners on the dressing aid in reduction of the risk of dressing lifting or snagging, which may reduce the frequency of dressing change by maintaining an optimal dressing condition



Office : Q-201, Netaji Building, Sion Mahim Link Road, Mahim(E), Mumbai 17, Maharashtra India, Ph +91 7506644100/200/300/400

Email: pinnacledrapes@hotmail.com | Website : www.pinnaclemediproducts.com

Factory : Unit 112 to 115 Bldg No. 28, HDIL Ind Park, Chandansar, Virar (E), Palghar Dist. Maharashtra, Pin 401303, India

- ✓ **Easy to apply:** The protector paper, which is slit across the width of the dressing, enables quick application without the risk of touching the absorbent pad or adhesive area with fingers or forceps. Easily openable packaging facilitates clean dressing technique
- ✓ **Hypoallergenic adhesive:** The hypoallergenic adhesive is spread evenly onto the non-woven backing layer of the dressing, providing safe and secure fixation over the wound site
- ✓ **ETO Sterilized**

Precautions

- Please consult the physician for any allergic responses of the patient
- Stop any bleeding at the site before applying the dressing
- Do not stretch the dressing during application as tension can cause skin trauma
- Make sure the skin is clean, free of soap residue and lotion, and allowed to dry thoroughly before applying the dressing to prevent skin irritation and to ensure good adhesion
- Do not use it if the packaging has been tampered with. Use of an un-sterile product may lead to infection or injury
- Do not re-sterilize
- For single use only. Discard after use



Available Sizes

Code No.	Size	Dressing per pack
PD 1040	5 x 7 cm	25
PD 1041	6 x 8.3 cm	25
PD 1042	10 x 10 cm	25
PD 1043	10 x 15 cm	10
PD 1044	10 x 20 cm	10
PD 1045	10 x 25 cm	10
PD 1046	10 x 30 cm	10

PINAPORE[®]-IV

NON WOVEN ISLAND ADHESIVE DRESSING WITH PAD

IV-DRESSING WITH PAD AND EXTRA STRIP



Product Description :

PINAPORE[®]-IV, Non-woven I.V. Dressing is made of non-woven backing material evenly coated with a polymeric adhesive, attached with an absorbent, non-adherent pad.

Application :

It provides a breathable and quick fixation to secure I.V. Cannula and other intramuscular catheters and percutaneous devices to the skin.

Features

- ✓ Simple and comfortable to use, painless removal
- ✓ Highly conformable
- ✓ Permeable to air and humidity, allowing skin to breathe and preventing infections due to moisture build up
- ✓ Hypoallergenic materials ensure no skin irritation

PINAPORE®-IV

NON WOVEN ISLAND ADHESIVE DRESSING WITH PAD

IV-DRESSING WITH PAD AND EXTRA STRIP

How To Use :

- ✓ Open the seal of the pouch and remove the dressing.
- ✓ Peel off the bottom liner exposing the pad and the adhesive surface.
- ✓ Position the dressing such that the pad covers the vein puncture site and the port is located at the neck between the two tabs.
- ✓ Remove the bottom liner to expose the adhesive surface of the tabs and press to hold the cannula on to the skin.
- ✓ Smoothen the dressing by pressing from centre towards the edges using firm pressure to enhance the adhesion.

How To Remove :

First peel off the tabs and free the cannula while supporting the cannula and the skin. This would prevent skin trauma or dislodgement of the cannula. Then peel off the remaining dressing. Pull the dressing in the opposite direction to facilitate painless removal.

Available Sizes

Code No.	Size	Dressing per pack
PD1050	6 x 8 cm	50






OPHEALS[®] - R

TRANSPARENT FILM DRESSING ROLL

OPHEALS[®] - R is a roll of transparent adhesive film that has been designed to allow easy application of the film to awkward areas of the body and over dressings and tubes. It is waterproof and provides excellent adherence, even while the patient is bathing.


- ✓ OPHEALS - R is a transparent, waterproof fixation for dressings.
- ✓ The film is highly conformable and extensible to increase patient comfort due to its shape and its unique hypoallergenic adhesive.
- ✓ It can be used for : Retention of primary dressings, low-adherent dressings.
- ✓ There is fixation of tubing and attachment of leg bags, stoma devices, etc.
- ✓ It can be cut to size.
- ✓ It is waterproof so you can shower whilst wearing the transparent film.
- ✓ It is easy to apply and remove and is a low friction film.
- ✓ It has a bacterial barrier that aids in the prevention of external bacteria contaminating the wound.

Available Sizes	Code No.	Size cm x mt	Dressing per pack
	PD1030	5 cm x 10 mtr	1 Roll
	PD1031	7.5 cm x 10 mtr	1 Roll
	PD1032	10 cm x 10 mtr	1 Roll
	PD1033	15 cm x 10 mtr	1 Roll

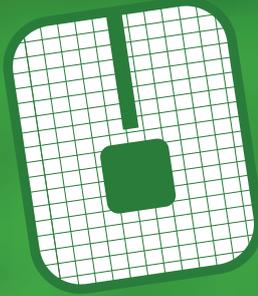

OPHEALS® - IV

Transparent Film Dressing with pad - IV Dressing

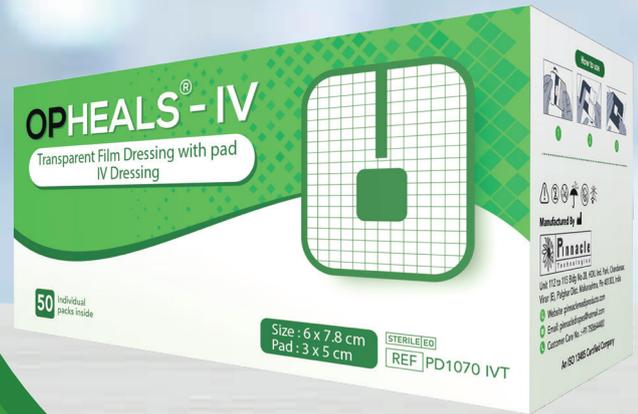

Product Description :

OPHEALS® IV, Transparent I.V. Dressing consists of a transparent medical film as a backing material, evenly coated with polymeric adhesive, attached with an absorbent, non-adherent pad.

Application:

For securing I.V. Cannula in cases with long term use. The transparent film allows regular inspection of the cannula insertion site for signs of infection or phlebitis.

Features:

- ✓ High water vapour permeability prevents moisture build up and thereby reduces risk of infection
- ✓ Minimises number of dressing changes as it keeps the skin drier, thereby improving dressing adherence
- ✓ Comfortable on skin due to conformable and hypoallergenic qualities
- ✓ Excellent cannula fixation stability allows patient to move freely with the cannula fixed
- ✓ Convenient regular visual inspection due to transparency of the film. Identification of infection or phlebitis possible without removing the dressing
- ✓ Easy and painless removal due to stretchability of the film



Office : Q-201, Netaji Building, Sion Mahim Link Road, Mahim(E), Mumbai 17, Maharashtra India, Ph +91 7506644100/200/300/400

Email: pinnacledrapes@hotmail.com | Website : www.pinnaclemediproducts.com

Factory : Unit 112 to 115 Bldg No. 28, HDIL Ind Park, Chandansar, Virar (E), Palghar Dist. Maharashtra, Pin 401303, India

How To Use:

- ✓ Open the seal of the pouch and remove the dressing
- ✓ Place the extra pad (optional) under the cannula for cushioning support
- ✓ Peel off the bottom liner exposing the pad and the adhesive surface
- ✓ Position the dressing such that the pad covers the vein puncture site and the port is located at the neck between the two tabs
- ✓ Remove the bottom liner to expose the adhesive surface of the tabs and press to hold the cannula on to the skin
- ✓ Remove the two top liners to expose the transparent top of the dressing
- ✓ Smoothen the dressing by pressing from centre towards the edges using firm pressure to enhance the adhesion

How To Remove:

First peel off the tabs and free the cannula while supporting the cannula and the skin. This would prevent skin trauma or dislodgement of the cannula. Then peel off the remaining dressing. Pull the dressing in the opposite direction to facilitate painless removal.

Available Sizes

Code No.	Size	Dressing per pack
PD1070 IV T	6 x 7 cm	50



OPHEALS[®] - PLUS

Transparent Film Adhesive Dressing with Pad



Product Description:

Transparent Film Adhesive Dressing is a sterile, hypoallergenic, conformable, post operative surgical dressing with a central absorbent and non adherent pad that helps in moist wound healing and painless dressing removal. This dressing is breathable, waterproof, low maintenance and acts as a bacterial and viral barrier thus protecting against infections.

Application:

This is used to provide protection for closed, clean surgical wounds, incisions, excisions, donor sites, etc. which have low to moderate exudates. It can also be used as a secondary dressing over hydrogels, alginates, gauze or other primary dressings or as a tracheostomy tube dressing, catheter/cannula site cover.



Features:

- ✓ **Microbial Barrier:** Bacterial and Viral barrier to prevent surgical site infections
- ✓ **Waterproof:** Waterproof barrier which allows patients to bathe or shower without removing the dressing
- ✓ **Highly absorbent pad:** Dressing consists of a highly absorbent pad that maximizes passage of blood and exudate into the dressing, minimizes the risk of fluid strikethrough
- ✓ **Conformable:** Conformable to adjust to natural shape of skin
- ✓ **Low adherent wound contact layer:** The pad has a low adherent wound contact layer which minimises disruption or trauma to the underlying tissue on removal of the dressing, reducing the risk of pain on removal

OPHEALS[®] - PLUS

Transparent Film Adhesive Dressing with Pad

- ✓ **Breathability:** Breathable film allows skin to function normally with good exchange of moisture vapor and oxygen
- ✓ **Rounded corners:** Rounded corners on the dressing aid in reduction of the risk of dressing lifting or snagging, which may reduce the frequency of dressing change by maintaining an optimal dressing condition
- ✓ **Easy to apply:** The protector paper, which is slit across the width of the dressing, enables quick application without the risk of touching the absorbent pad or adhesive area with fingers or forceps. Easily openable packaging facilitates clean dressing technique
- ✓ **Hypoallergenic adhesive:** The hypoallergenic adhesive is spread evenly onto the non-woven backing layer of the dressing, providing safe and secure fixation over the wound site
- ✓ **ETO Sterilized**

Precautions:

- ✓ Please consult the physician for any allergic responses of the patient.
- ✓ Stop any bleeding at the site before applying the dressing.
- ✓ Do not stretch the dressing during application as tension can cause skin trauma.
- ✓ Make sure the skin is clean, free of soap residue and lotion, and allowed to dry thoroughly before applying the dressing to prevent skin irritation and to ensure good adhesion.
- ✓ Do not use it if the packaging has been tampered with. Use of an un-sterile product may lead to infection or injury.
- ✓ Do not re-sterilize.
- ✓ For single use only. Discard after use.

Available Sizes

Code No.	Size	Dressing per pack
PD 1020	5 x 7 cm	25
PD 1021	6 x 8.3 cm	25
PD 1022	10 x 10 cm	25
PD 1023	10 x 15 cm	10
PD 1024	10 x 20 cm	10
PD 1025	10 x 25 cm	10
PD 1026	10 x 30 cm	10